

REF			SYSTEM
09043284190	09043284500	300	<b>cobas e 402</b> <b>cobas e 801</b>

## English

### System information

Short name	ACN (application code number)
FT4 4	10195

### Intended use

Immunoassay for the in vitro quantitative determination of free thyroxine in human serum and plasma.

The electrochemiluminescence immunoassay "ECLIA" is intended for use on **cobas e** immunoassay analyzers.

### Summary

Thyroxine (T<sub>4</sub>) is the main thyroid hormone secreted into the bloodstream by the thyroid gland. Together with triiodothyronine (T<sub>3</sub>) it plays a vital role in regulating the body's metabolic rate, influences the cardiovascular system, growth and bone metabolism, and is important for normal development of gonadal functions and nervous system.<sup>1</sup>

T<sub>4</sub> circulates in the bloodstream as an equilibrium mixture of free and serum proteins bound hormone. Free T<sub>4</sub> (fT<sub>4</sub>) is the unbound and biologically active form, which represents only 0.03 % of the total T<sub>4</sub>. The remaining T<sub>4</sub> is inactive and bound to serum proteins such as thyroxine binding globulin, TBG (75 %), pre-albumin (15 %), and albumin (10 %).<sup>2,3,4,5</sup>

The determination of free T<sub>4</sub> has the advantage of being independent of changes in the concentrations and binding properties of these binding proteins; additional determination of a binding parameter (T-uptake, TBG) is therefore unnecessary. Thus free T<sub>4</sub> is a useful tool in clinical routine diagnostics for the assessment of the thyroid function. It should be measured together with TSH if thyroid disorders are suspected and is also suitable for monitoring thyrosuppressive therapy.<sup>1,6,7</sup>

A variety of methods are available for estimating the free thyroid hormone levels. The direct measurement of fT<sub>4</sub> and fT<sub>3</sub> via equilibrium dialysis or ultrafiltration is mainly used as a reference method for standardizing, the immunological procedures generally used for routine diagnostic purposes.<sup>6,7</sup>

In the Elecsys FT4 IV assay a T<sub>4</sub>-specific antibody labeled with a ruthenium complex<sup>a)</sup> is used to determine the free thyroxine.

a) Tris(2,2'-bipyridyl)ruthenium(II)-complex (Ru(bpy)<sub>3</sub><sup>2+</sup>)

### Test principle

Competition principle. Total duration of assay: 18 minutes.

- 1st incubation: 9 µL of sample and a T<sub>4</sub>-specific antibody labeled with a ruthenium complex.
- 2nd incubation: After addition of biotinylated T<sub>4</sub> and streptavidin-coated microparticles, the still-free binding sites of the labeled antibody become occupied, with formation of an antibody-hapten complex. The entire complex becomes bound to the solid phase via interaction of biotin and streptavidin.
- The reaction mixture is aspirated into the measuring cell where the microparticles are magnetically captured onto the surface of the electrode. Unbound substances are then removed with ProCell II M. Application of a voltage to the electrode then induces chemiluminescent emission which is measured by a photomultiplier.
- Results are determined via a calibration curve which is instrument-specifically generated by 2-point calibration and a master curve provided via the **cobas** link.

### Reagents - working solutions

The **cobas e** pack is labeled as FT4 4.

- M Streptavidin-coated microparticles, 1 bottle, 13.2 mL:  
Streptavidin-coated microparticles 0.72 mg/mL; preservative.
- R1 Anti-T<sub>4</sub>-Ab-Ru(bpy)<sub>3</sub><sup>2+</sup>, 1 bottle, 19.7 mL:  
Monoclonal anti-T<sub>4</sub>-antibody (rabbit) labeled with ruthenium complex 75 ng/mL; phosphate buffer 100 mmol/L, pH 7.0; preservative.

- R2 T<sub>4</sub>-biotin, 1 bottle, 19.7 mL:  
Biotinylated T<sub>4</sub> 2.5 ng/mL; phosphate buffer 100 mmol/L, pH 7.0; preservative.

### Precautions and warnings

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal.

Safety data sheet available for professional user on request.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



### Warning

- H317 May cause an allergic skin reaction.

### Prevention:

- P261 Avoid breathing dust/fume/gas/mist/vapours/spray.
- P272 Contaminated work clothing should not be allowed out of the workplace.
- P280 Wear protective gloves.

### Response:

- P333 + P313 If skin irritation or rash occurs: Get medical advice/attention.
- P362 + P364 Take off contaminated clothing and wash it before reuse.

### Disposal:

- P501 Dispose of contents/container to an approved waste disposal plant.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

### Reagent handling

The reagents in the kit have been assembled into a ready-for-use unit that cannot be separated.

All information required for correct operation is available via the **cobas** link.

### Storage and stability

Store at 2-8 °C.

Do not freeze.

Store the **cobas e** pack **upright** in order to ensure complete availability of the microparticles during automatic mixing prior to use.

Stability:	
unopened at 2-8 °C	up to the stated expiration date
on the analyzers	16 weeks

### Specimen collection and preparation

Only the specimens listed below were tested and found acceptable.

# Elecsys FT4 IV

Serum collected using standard sampling tubes or tubes containing separating gel.

Li-heparin, K<sub>2</sub>-EDTA and K<sub>3</sub>-EDTA plasma.

Plasma tubes containing separating gel can be used.

Criterion: Slope 0.9-1.1 + intercept within  $\pm 0.6$  pmol/L + coefficient of correlation  $\geq 0.95$ .

Stable for 5 days at 20-25 °C, 7 days at 2-8 °C, 30 days at -20 °C ( $\pm 5$  °C). Freeze only once.

The sample types listed were tested with a selection of sample collection tubes that were commercially available at the time of testing, i.e. not all available tubes of all manufacturers were tested. Sample collection systems from various manufacturers may contain differing materials which could affect the test results in some cases. When processing samples in primary tubes (sample collection systems), follow the instructions of the tube manufacturer.

Centrifuge samples containing precipitates before performing the assay.

Do not use heat-inactivated samples.

Do not use samples and controls stabilized with azide.

Ensure the samples and calibrators are at 20-25 °C prior to measurement.

Due to possible evaporation effects, samples and calibrators on the analyzers should be analyzed/measured within 2 hours.

## Materials provided

See "Reagents – working solutions" section for reagents.

## Materials required (but not provided)

- [REF] 09043292190, CalSet FT4 IV, 4 x 1.0 mL
- [REF] 11731416190, PreciControl Universal, for 4 x 3.0 mL
- General laboratory equipment
- **cobas e** analyzer

Additional materials for **cobas e** 402 and **cobas e** 801 analyzers:

- [REF] 06908799190, ProCell II M, 2 x 2 L system solution
- [REF] 04880293190, CleanCell M, 2 x 2 L measuring cell cleaning solution
- [REF] 07485409001, Reservoir Cup, 8 cups to supply ProCell II M and CleanCell M
- [REF] 06908853190, PreClean II M, 2 x 2 L wash solution
- [REF] 05694302001, Assay Tip/Assay Cup tray, 6 magazines x 6 magazine stacks x 105 assay tips and 105 assay cups, 3 wasteliners
- [REF] 07485425001, Liquid Flow Cleaning Cup, 2 adaptor cups to supply ISE Cleaning Solution/Elecsys SysClean for Liquid Flow Cleaning Detection Unit
- [REF] 07485433001, PreWash Liquid Flow Cleaning Cup, 1 adaptor cup to supply ISE Cleaning Solution/Elecsys SysClean for Liquid Flow Cleaning PreWash Unit
- [REF] 11298500316, ISE Cleaning Solution/Elecsys SysClean, 5 x 100 mL system cleaning solution

## Assay

For optimum performance of the assay follow the directions given in this document for the analyzer concerned. Refer to the appropriate operator's manual for analyzer-specific assay instructions.

Resuspension of the microparticles takes place automatically prior to use.

Place the cooled (stored at 2-8 °C) **cobas e** pack on the reagent manager. Avoid foam formation. The system automatically regulates the temperature of the reagents and the opening/closing of the **cobas e** pack.

## Calibration

Traceability: This method was standardized against the Elecsys FT4 III method. The Elecsys FT4 III assay is traceable to the Elecsys FT4 II assay which in turn is traceable to the Enzymun-Test FT4 which has been standardized using equilibrium dialysis.<sup>5,8</sup>

The predefined master curve is adapted to the analyzer using the relevant CalSet.

**Calibration frequency:** Calibration must be performed once per reagent lot using fresh reagent (i.e. not more than 24 hours since the **cobas e** pack was registered on the analyzer).

Calibration interval may be extended based on acceptable verification of calibration by the laboratory.

Renewed calibration is recommended as follows:

- after 12 weeks when using the same reagent lot
- after 28 days when using the same **cobas e** pack on the analyzer
- as required: e.g. quality control findings outside the defined limits

## Quality control

For quality control, use PreciControl Universal.

In addition, other suitable control material can be used.

Controls for the various concentration ranges should be run individually at least once every 24 hours when the test is in use, once per **cobas e** pack, and following each calibration.

The control intervals and limits should be adapted to each laboratory's individual requirements. Values obtained should fall within the defined limits. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

If necessary, repeat the measurement of the samples concerned.

Follow the applicable government regulations and local guidelines for quality control.

## Calculation

The analyzer automatically calculates the analyte concentration of each sample either in pmol/L, ng/dL or ng/L.

Conversion factors:      pmol/L x 0.077688 = ng/dL  
    ng/dL x 12.872 = pmol/L  
    pmol/L x 0.77688 = ng/L

## Limitations - interference

The effect of the following endogenous substances and pharmaceutical compounds on assay performance was tested. Interferences were tested up to the listed concentrations and no impact on results was observed.

### Endogenous substances

Compound	Concentration tested
Bilirubin	$\leq 701$ $\mu$ mol/L or $\leq 41$ mg/dL
Hemoglobin	$\leq 0.621$ mmol/L or $\leq 1000$ mg/dL
Intralipid	$\leq 2000$ mg/dL
Biotin	$\leq 4912$ nmol/L or $\leq 1200$ ng/mL
Rheumatoid factors	$\leq 1200$ IU/mL
IgG	$\leq 7$ g/dL
IgA	$\leq 1.6$ g/dL
IgM	$\leq 1$ g/dL

Criterion: Recovery of  $\leq \pm 0.6$  pmol/L of initial value  $\leq 6$  pmol/L and  $\pm 10$  % of initial value  $> 6$  pmol/L.

Any influence that might affect the binding behavior of the binding proteins can alter the result of the FT4 tests (e.g. drugs, NTIs (Non-Thyroid-Illness) or patients suffering from FDH (Familial Dysalbuminemic Hyperthyroxinemia)).<sup>9,10</sup>

The test cannot be used in patients receiving treatment with lipid-lowering agents containing D-T4. If the thyroid function is to be checked in such patients, the therapy should first be discontinued for 4-6 weeks to allow the physiological state to become re-established.<sup>11</sup>

Autoantibodies to thyroid hormones can interfere with the assay.<sup>7</sup>

### Pharmaceutical substances

In vitro tests were performed on 14 commonly used pharmaceuticals. No interference with the assay was found. For the common pharmaceuticals phenylbutazone, itraconazole and ibuprofen no interference was observed for concentrations  $\leq 80$   $\mu$ g/mL,  $\leq 15$   $\mu$ g/mL and  $\leq 109$   $\mu$ g/mL, respectively.

In addition, the following special thyroid drugs were tested. For the stated concentrations, no interference with the assay was found. The drugs furosemide, carbamazepine, phenytoin and levothyroxine sodium (L-T4, synthetic levothyroxine<sup>12</sup>) caused elevated FT4 recovery at the daily therapeutic concentration.



# Elecsys FT4 IV

Passing/Bablok<sup>13</sup>

$$y = 1.048x - 0.184$$

$$r = 0.978$$

Linear regression

$$y = 1.065x - 0.402$$

$$r = 0.999$$

The sample concentrations were between 1.32 and 93.3 pmol/L.

## Analytical specificity

The following cross-reactivities were found, tested with FT4 concentrations of approximately 19.4 pmol/L and 64.5 pmol/L:

Cross-reactant	Concentration tested ng/dL	Cross-reactivity %
L-T3	50000	0.005
D-T3	50000	0.003
rT3	190000	0.002
3-iodo-L-tyrosine	10000000	0.000
3,5-diiodo-L-tyrosine	10000000	0.000
3,3',5-triiodothyroacetic acid	100000	0.000
3,3',5,5'-tetraiodothyroacetic acid	100000	0.003

## References

- Kronenberg HM, Melmed S, Polonsky KS, et al. *Williams Textbook of Endocrinology*. Saunders Elsevier, Philadelphia, 12th edition, 2011, chapter 10, p. 301-311.
- Robbins J, Rall JE. The interaction of thyroid hormones and protein in biological fluids. *Recent Prog Horm Res* 1957;13:161-208.
- Oppenheimer JH. Role of plasma proteins in the binding, distribution and metabolism of the thyroid hormones. *N Engl J Med* 1968;278(21):1153-1162.
- DeGroot LJ, Larsen PR, Hennemann G. Transport of thyroid hormone and cell uptake. *The thyroid and its diseases*. Wiley and Sons, New York, 1984:62-65.
- Ekens RP. Measurement of free hormones in blood. *Endocr Rev* 1990;11(1):5-46.
- Wu AHB. *Tietz Clinical Guide To Laboratory Tests*. Saunders Elsevier, Philadelphia, 4th edition, 2006, section II, p. 1046-1048.
- Brent GA. *Thyroid Function Testing*. Springer, Berlin, 1st edition, 2010, chapter 5, p. 86-101.
- Ekens RP, Ellis SM. The radioimmunoassay of free thyroid hormones in serum. In Robbins J, Braverman LE (eds). *Thyroid research, Proceedings of the Seventh International Thyroid Conference*, Boston. Amsterdam, Excerpta Medica 1975:597.
- Wada N, Chiba H, Shimizu C, et al. A novel missense mutation in codon 218 of the albumin gene in a distinct phenotype of familial dysalbuminemic hyperthyroxinemia in a Japanese kindred. *J Clin Endocrinol Metab* 1997;82(10):3246-3250.
- Arevalo G. Prevalence of familial dysalbuminemic hyperthyroxinemia in serum samples received for thyroid testing. *Clin Chem* 1991;37(8):1430-1431.
- Bantle JP, Hunninghake DB, Frantz ID, et al. Comparison of effectiveness of thyrotropin-suppressive doses of D- and L-thyroxine in treatment of hypercholesterolemia. *Am J Med* 1984;77(3):475-481.
- Hennessey JV. The emergence of levothyroxine as a treatment for hypothyroidism. *Endocrine* 2017;55(1):6-18.
- Bablok W, Passing H, Bender R, et al. A general regression procedure for method transformation. Application of linear regression procedures for method comparison studies in clinical chemistry, Part III. *J Clin Chem Clin Biochem* 1988 Nov;26(11):783-790.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets and the Method Sheets of all necessary components (if available in your country).

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

## Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see [dialog.roche.com](http://dialog.roche.com) for definition of symbols used):

	Contents of kit
	Analyzers/Instruments on which reagents can be used
	Reagent
	Calibrator
	Volume for reconstitution
	Global Trade Item Number

COBAS, COBAS E, ELECSYS and PRECICONTROL are trademarks of Roche. INTRALIPID is a trademark of Fresenius Kabi AB.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2021, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim  
[www.roche.com](http://www.roche.com)

+800 5505 6606

